EN Dental reanatomization to maximize the appearance of the smile

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30-year-old male patient.

COMPLAINT

Incisal edge fracture of the upper central incisor and dissatisfaction with the size of central incisors.

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EN CLINICAL/RADIOGRAPHIC EXAM

The incisor edges of the upper central incisors showed unsatisfactory restorations and an evident fracture line in the angle between the restoration and the tooth remnant.

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TREATMENT

A restorative treatment was planned using the BRB technique (Bertolho, Ricci, Barrotte matrix)¹ with composites, to perform the restoration and re-anatomization of the upper incisors in a single session. The advantage of this technique is that it does not require the use of study models and diagnostic waxing to design a matrix to make the palatal wall.

The first step was to choose the shade by means of cross-polarized light through photography with polarized filters². This method allows suppressing reflections of ambient light and the flash of the photography equipment, removing brightness to see only the hue and chroma. After that, the BRB matrix

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was made, followed by the removal of the restorations and adaptation of the remnant. The restoration was then performed with Opallis composite in order to improve shade and size of the upper central incisors.

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STEP BY STEP | PASO A PASO

1. Front view, initial situation. It presents a fracture of the incisor edges.

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2. Shade selection with increments of Opallis in shades: EA1 (enamel), VH, DA1 (dentin), Opaque Pearl and T-blue.

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3. Preparation of the BRB matrix, which is made by molding directly in the mouth with condensation silicone, which is positioned on the palatine faces, encompassing the incisor edges from premolar to premolar.

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4. Performing the BRB matrix technique by delimiting the incisal edges with pencils to be sculpted with a minicut drill (Verdent, Poland) in order to simulate the final palatine anatomy of the teeth to be restored.

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5. Adaptation of the BRB matrix after absolute isolation. Note the removal of the restorations and remnant preparation with a discrete bevel (3131 diamond tip).

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6. Hybridization of the dental remnant (etching with phosphoric acid Condac 37 and application of Ambar adhesive, following the manufacturer's recommendations) and light curing.

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7. Making the first layer of composite with Opaque Pearl shade (Opallis) by using the BRB matrix.

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8. Final appearance of the palatal wall after excesses removal.

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9. Application of a thin layer of enamel shade Opallis (EA1) to the mamelon region after the dentin shade Opallis (DA1) was inserted in the middle third covering half the thickness of the bevel.

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10. Characterization of the incisal edge by using T-blue effect shade (Opallis).

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12. Checking restorations size and shape with the BRB matrix.

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// Incisal edge fracture: a simple procedure to solve when there are good techniques (BRB Matrix) and FGM restorative materials for the achievement of satisfactory results. //

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16. Side view, enamel/composite interface appearance prior to the removal of excesses with multi-laminate drill.

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17. Use of disks of different grain sizes for finishing refinement.

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20. Final appearance of restorations after polishing with abrasive rubber, Diamond Flex disks and Diamond Excel polishing paste, reaching shine and smoothness similar to the patient's natural enamel.

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RESINA FOTOCURABLE

Opallis

5TH GENERATION LIGHT-CURING ADHESIVE

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