EN Natural appearance with **COMPOSITES**

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Ruptura del incisivo central derecho.

EN CLINICAL/RADIOGRAPHIC EXAM

Fracture of tooth 11, compromising the incisal edge. Vitality test and periapical radiography were performed and no alterations were found.

TREATMENT

Firstly, the case planning was carried out through photographs and waxing. Shade taking was performed by using the Polar Eyes filter. It was decided to proceed with the restorative treatment with nanohybrid composite resin (Opallis), using the stratification technique with dentin, enamel and effect shades, with the purpose to obtain a natural result.

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STEP BY STEP | PASO A PASO

1. Initial face photograph.
2. Intraoral photograph.
3. After molding, waxing of the plaster model was performed.

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4 and 5. We started the restorative sequence with previous anesthesia, absolute isolation and preparation of the restoration with Diamond Pro sandpaper polishing disks eliminate all sharp angles.

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6. Application of 37% phosphoric acid (Condac 37) for 15 seconds to both enamel and dentin.

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9 and 10. In the second, third and fourth layers of the restoration we used the dentin shade DA1 (Opallis) in order to reproduce the mamelons and leaving space for the translucent edge.

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12. Finally, we finished stratification with the enamel layer with shade EA1 (Opallis), reproducing the primary anatomy of the right central incisor.

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13. For finishing and polishing, we used Diamond Pro disks with the assistance of anatomical lines, polishing rubber and felt disks combined with the Diamond polishing paste.

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LIGHT-CURING NANOHYBRID COMPOSITE
Opallis

RESINA NANOHÍBRIDA FOTOCURABLE
Opallis

5th GENERATION LIGHT-CURING ADHESIVE
Ambar

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LIGHT-CURING GINGIVAL BARRIER
Top dam

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